


# MEDACURE

## BARIATRIC FREE STANDING TRAPEZE


### ASSEMBLY AND OPERATION MANUAL

#### MODEL # TR500

1. Trapeze installation as per Figure # 1

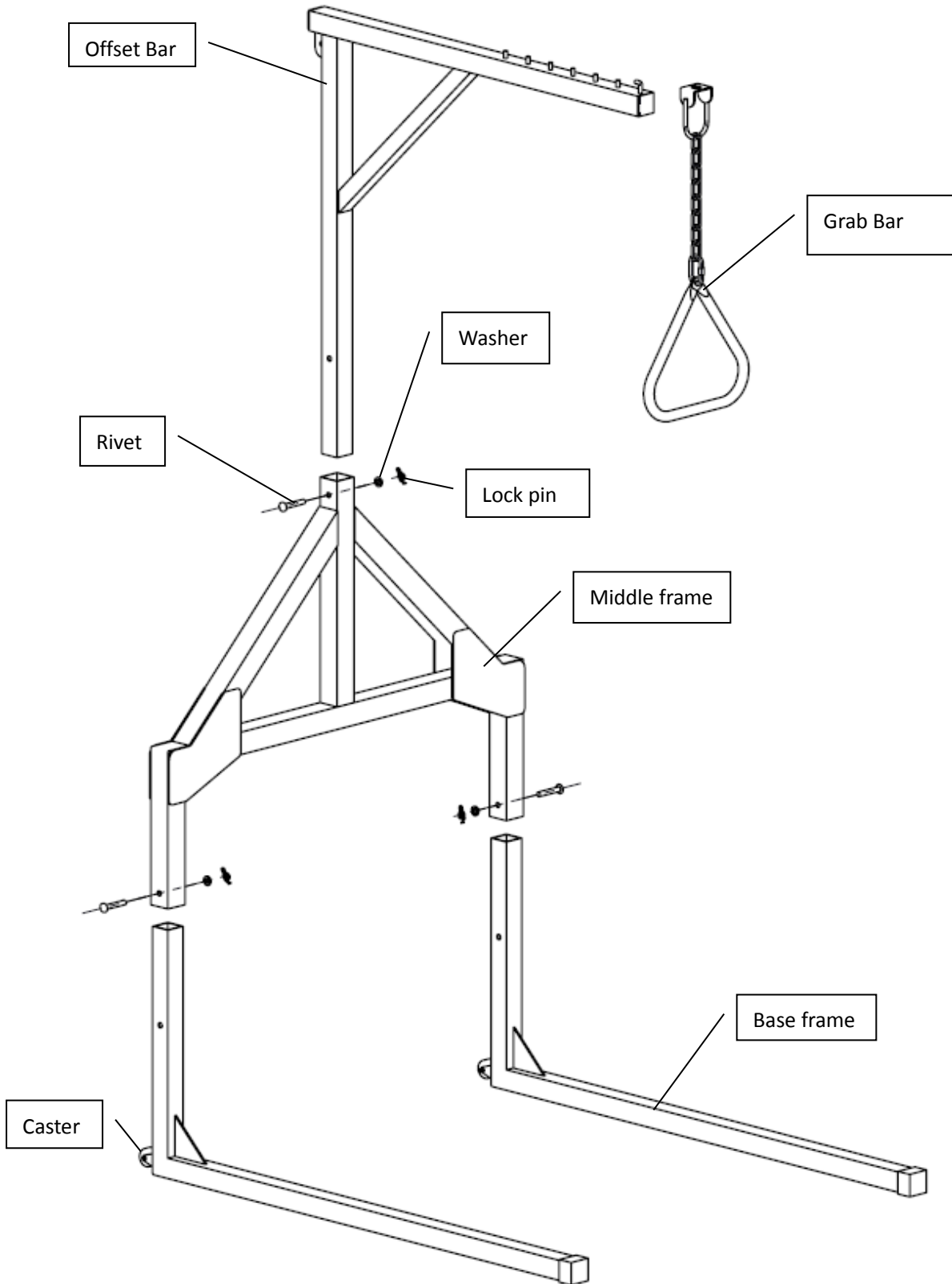

DISASSEMBLY

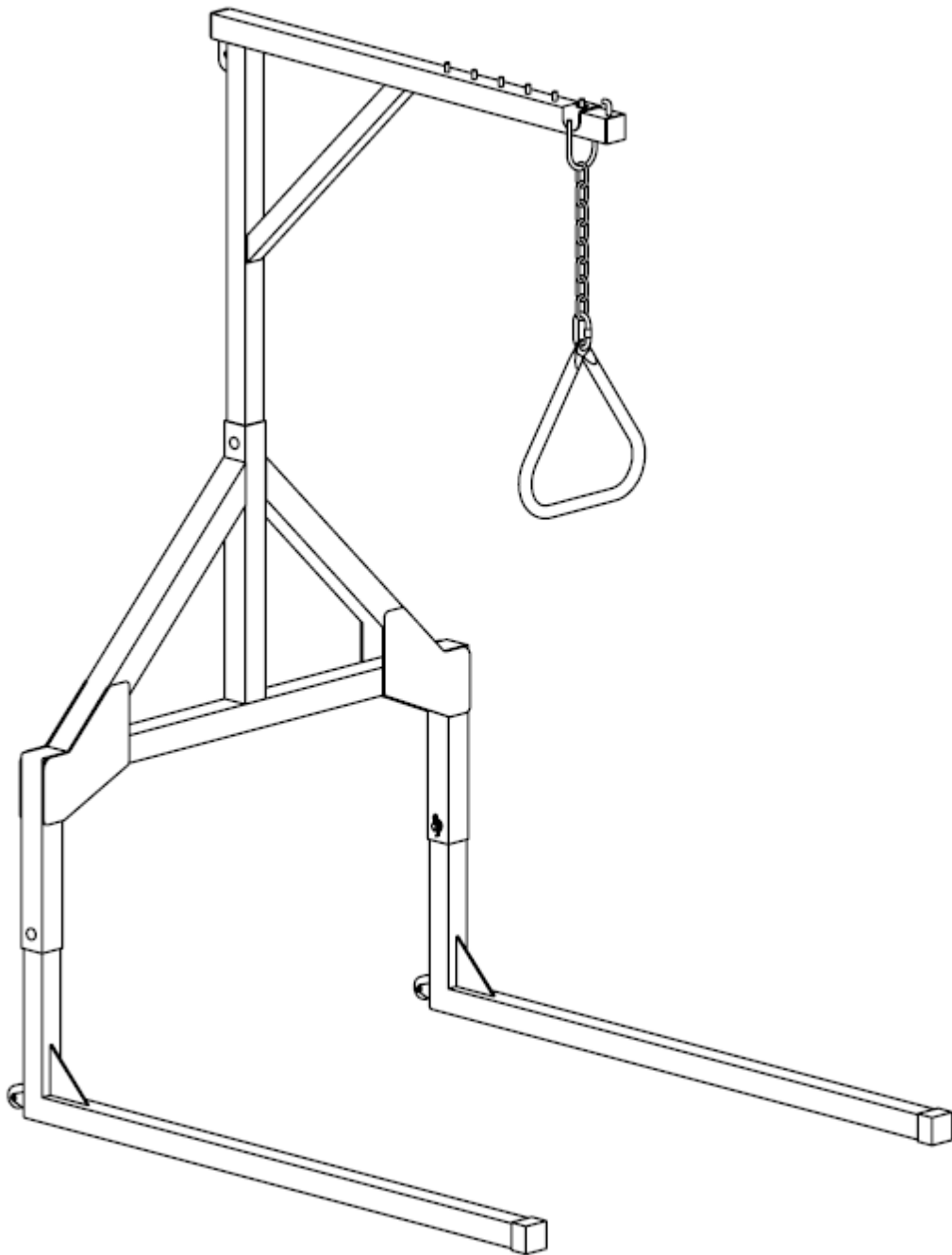


**ASSEMBLY**

**Figure # 1**

**2. Using the Trapeze**

**2.1 Moving the Trapeze**


There are 2 casters on the base of the frame to assist in moving the Trapeze. Slightly incline the Trapeze to ensure the casters touch the ground, gently push/pull to move as per Figure # 2.

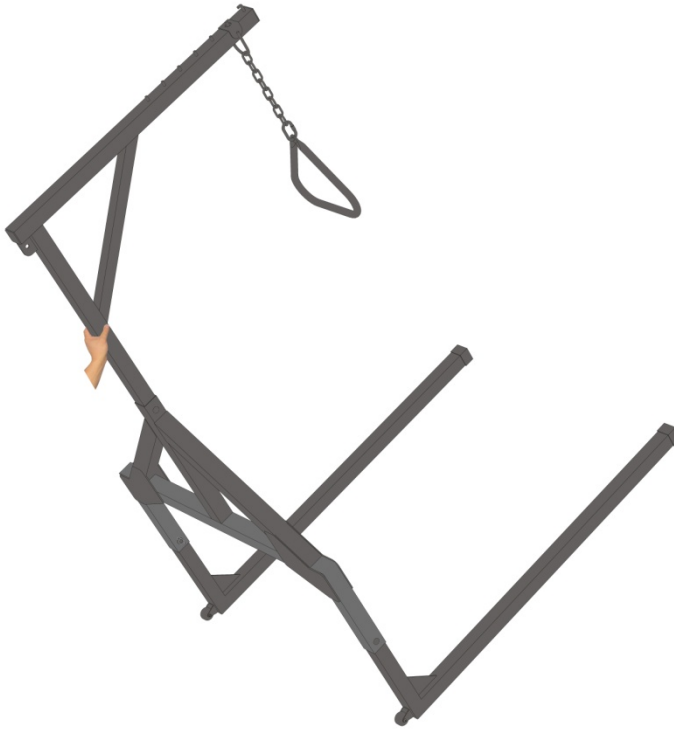

**Figure # 2**



## 2.2 Adjusting the position of the Grab Bar

There are 6 location bars on the top tube for adjusting the position as per Figure # 3

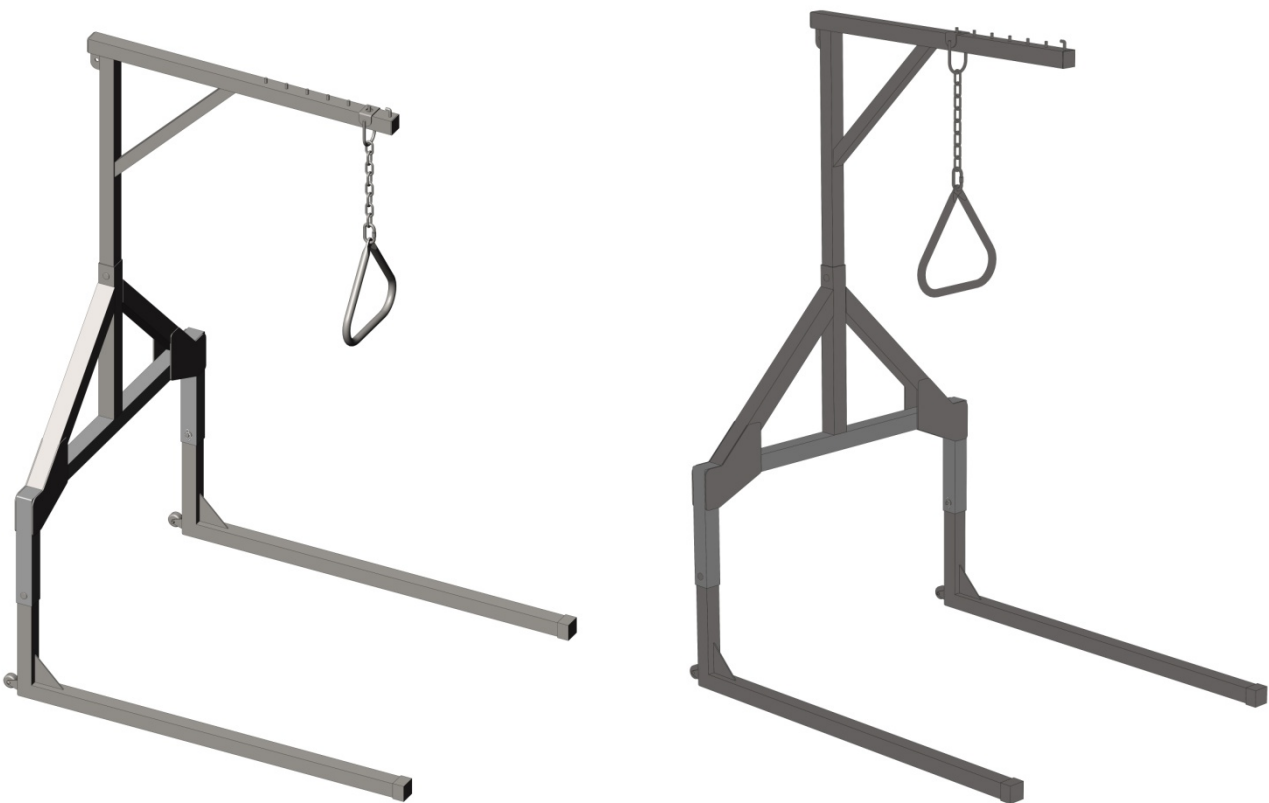

**Figure # 3**


The chain has 8 links to allow you to adjust the height of the Grab Bar as per Figure # 4

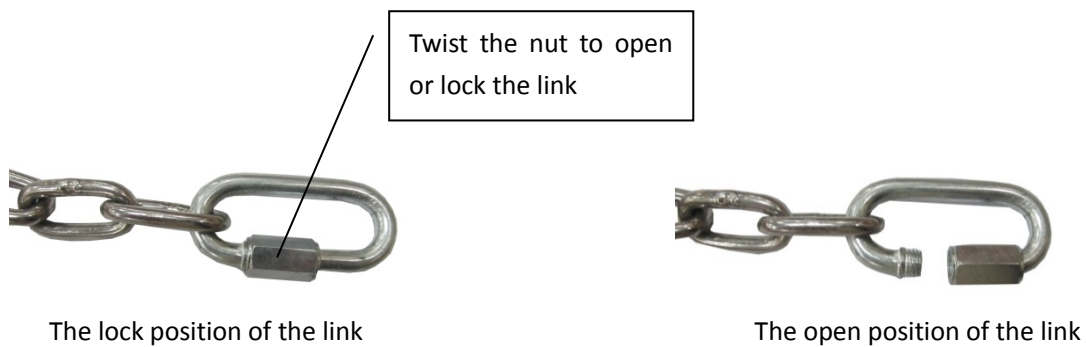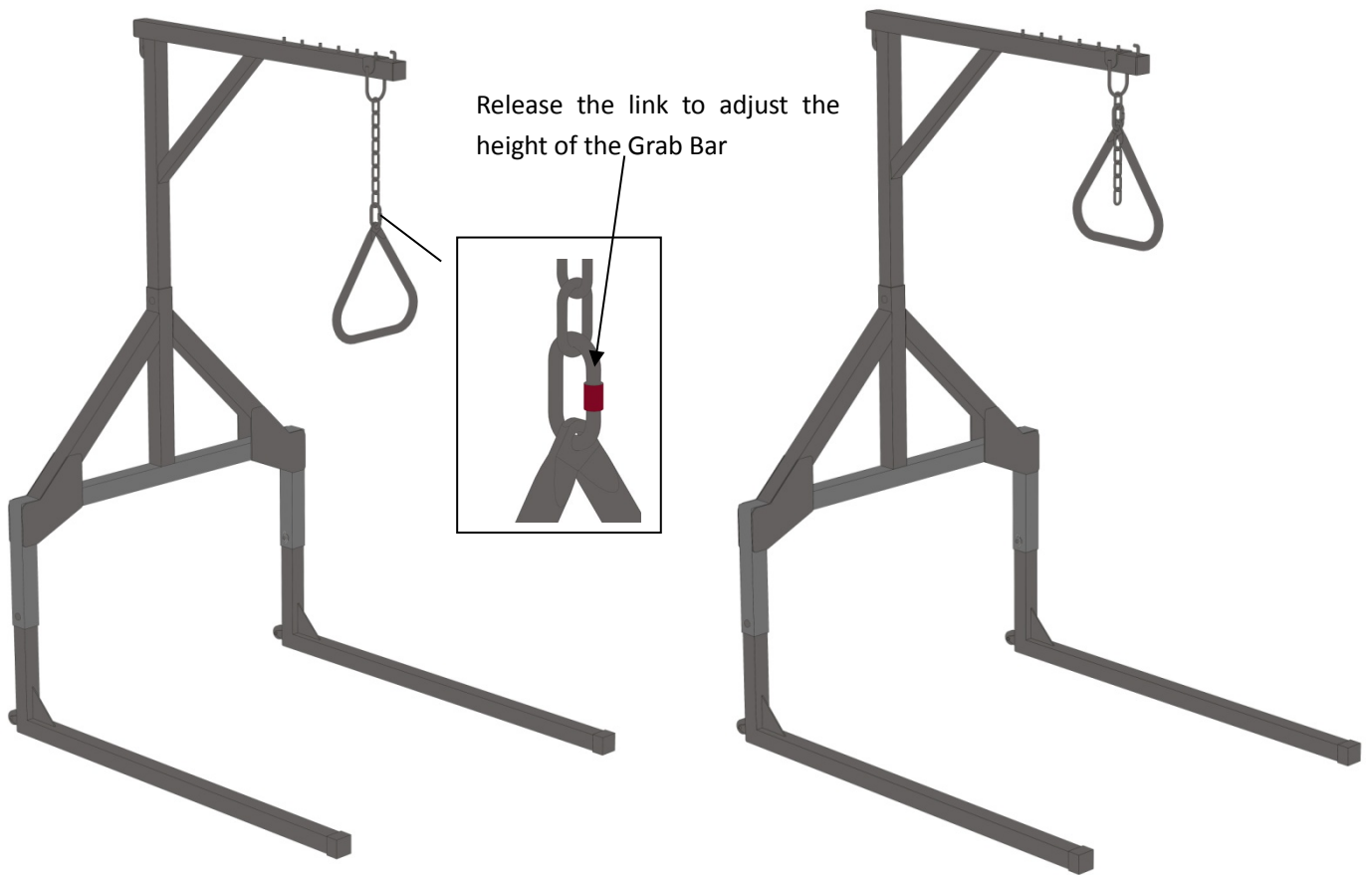



The lock position of the link

The open position of the link

Figure # 4

- 

**WARNING**  
**THE TRAPEZE WILL ASSIST AN INDIVIDUAL WEIGHING UP TO 500 LBS.** THE TRAPEZE IS DESIGNED TO PROVIDE SUPPORT, INCREASE STABILITY AND ASSIST THE USER WHEN REPOSITIONING IN BED. IT IS NOT DESIGNED TO SUPPORT THE TOTAL BODY WEIGHT OF THE PATIENT. THIS DEVICE IS NOT DESIGNED TO MOVE A PATIENT OR RESIDENT IN AND OUT OF BED. THIS DEVICE SHOULD BE USED FOR ASSISTANCE ONLY.
- 

**WARNING**  
 NO PART OR COMPONENT OF THE TR500 MEDACURE TRAPEZE SHOULD BE USED WITH NON-MEDACURE BEDS, LIFTS OR TRAPEZES.
- 

**WARNING**  
 AFTER ASSEMBLY AND BEFORE USE, MAKE SURE ALL HARDWARE IS SECURELY TIGHTENED.