

## QUARTER LENGTH SIDE RAILS - MODEL # HSR28

### Side Rails - Installation and Operation

Unscrew the star knob on the side rail mounting base. Insert the mounting base on to the 30 x

20 rectangular tube of the head deck. Adjust the mounting base well ensuring that it is positioned upright and straight and that it is clamped fully on the top and bottom of the rectangle deck tube. Tighten the black star knob until the rail is sturdy and does not shake - see Figure # 1 and 2.

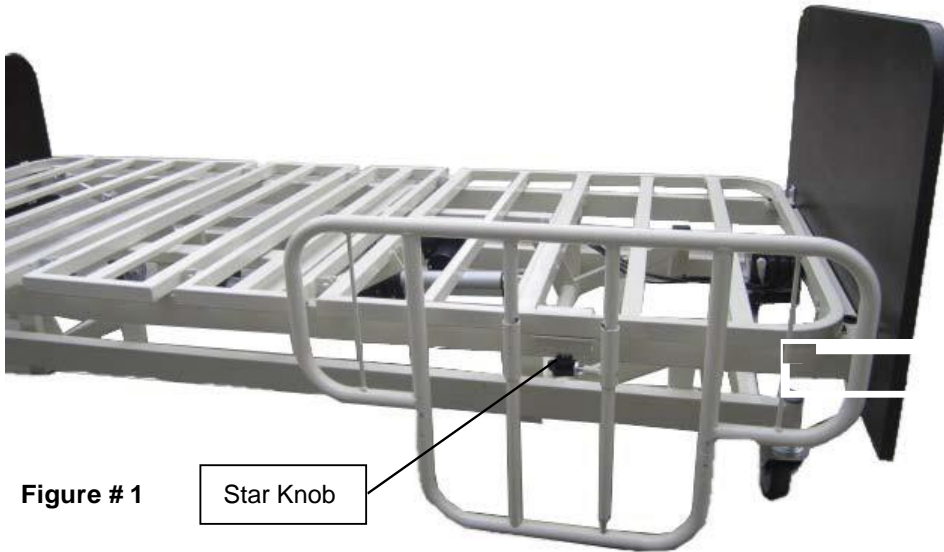


Figure # 1

Star Knob

### WARNING - IMPORTANT

While tightening the black star knob, ensure the silver metal mounting plate remains flat. Tighten the star knob on to the silver metal mounting plate until tight. Test the rail to confirm it is tight and does not shake. See Figure # 2 below.

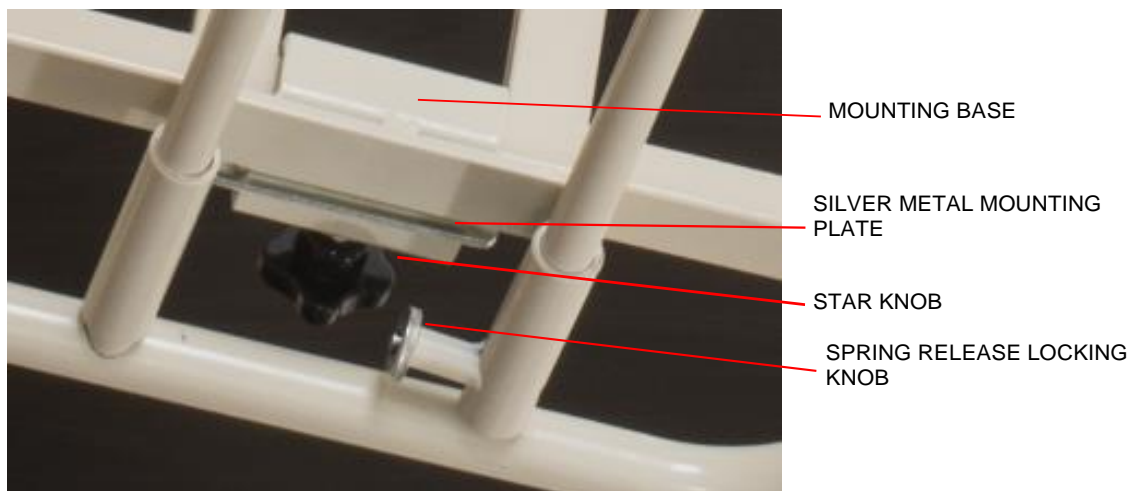


Figure # 2

### TO LOWER AND RAISE THE SIDE RAILS

Raise the side rail until it clicks into place. Pull the **spring release locking knob** button to lower or raise the side rail. Release to lock rail in place - see figure # 3.

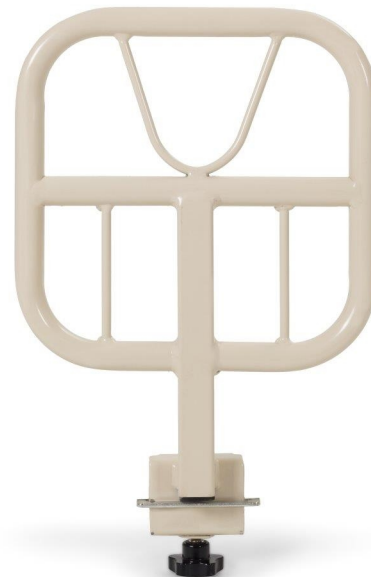


Figure # 14

---

### QBAR - PATIENT ASSIST GRAB BAR - MODEL # QB900

The QBar is a Patient Assist Grab Bar that clamps on to the head deck and locks in place with a star knob. To attach see instructions above for our standard Side Rails.



For rail entrapment guidelines please visit the FDA.Gov website

<https://www.fda.gov/medicaldevices/deviceregulationandguidance/guidancedocuments/ucm072662.htm>